

材质: 105g铜版纸


尺寸: 105 x 150 mm

印刷: 单黑双面印刷

装订方式: 骑马钉

封面封底内页留白, 最

后多出一张为空白页

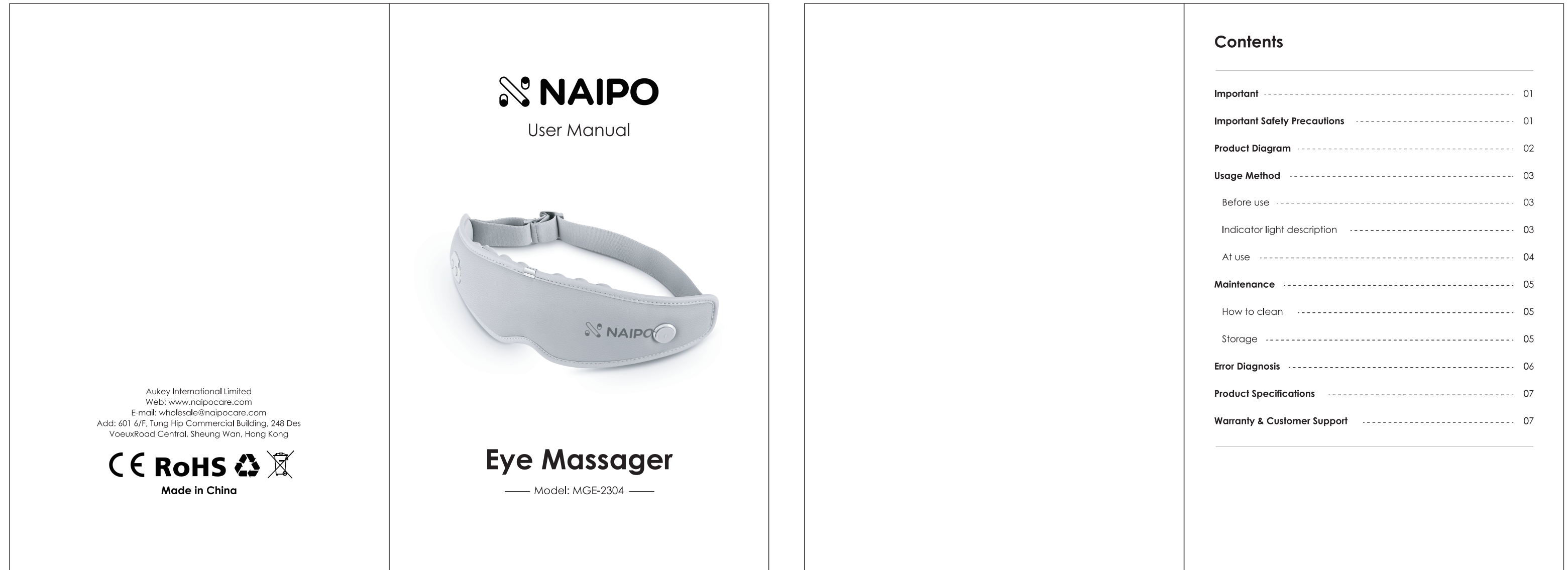


**Important**  
Please keep this manual properly for future reference. Before using this appliance, please read all the instructions carefully.

**Important Safety Precautions**

- Do not wear accessories around your eyes when using this product.
- For those with symptoms of eye disease, patients with major eye diseases such as eye surgery, glaucoma, cataract, and retinal detachment, please DO NOT use this product.
- For those who have medical electronic devices implants, such as cardiac pacemakers, or patients with malignant tumors, heart disease, or severe hypertension, please consult your physician before using this product.
- Children and mentally/physically disabled can only use the product with special supervision of a responsible adult.
- If you experience bodily pain and discomfort during use, please stop using it immediately and consult your doctor.
- This product is not a toy and can not be given to children for fun.
- This product is for single-person use. Do not operate simultaneously on more than 1 person each time.
- This product is only for home use. It is not a piece of medical equipment and is not intended for self-diagnosis, treatment, or other commercial purposes.
- Please charge the product using only accompanied type-c cable and please be careful while you charge.
- Please cut off the power when the product exhibit malfunctioning. Do not disassemble or fix the product on your own.
- Do not use the product if you have a fever or body temperature higher than 38°C.
- Please do not disassemble the product on your own to prevent product malfunctioning.
- Do not use a power source other than the rated power to prevent fire, physical injuries, or product damages.
- Do not damage, destroy, over-bend, pull, twist, or tie the type-c cable.
- Do not expose the product in the environment of high-temperature or hot water.
- The product should not be discarded with regular home disposal. Please follow the local guideline for disposing of the product.
- The direction of connecting the product or the type-c cable depends on the product model. Please do not force to connect to the product in case of damaged or shorted connected that may lead to fire or product malfunctions.

1

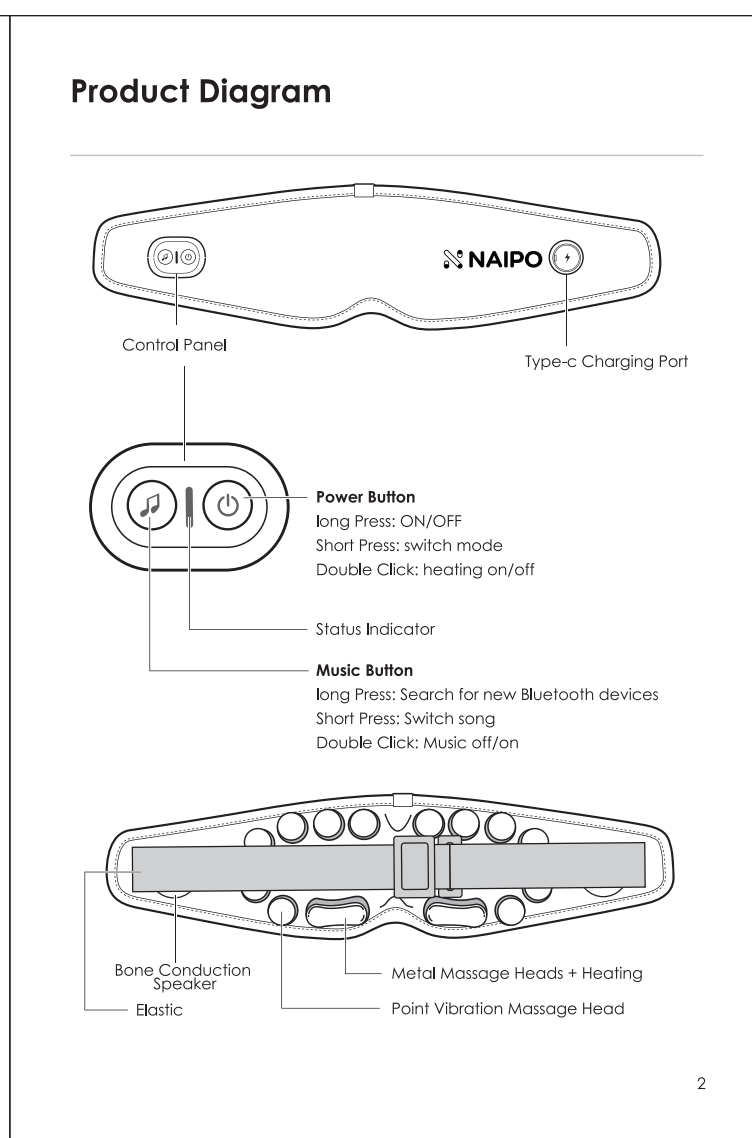

**Usage Method**

**Before use**

- Please check the integrity of the machine and accessories
- Please remove the accessories around the eyes during use to avoid affecting the massage area.
- Adjust the elastic headband to the size that fits your head shape.

**Indicator light description**

Power indicator light			
	Colour	Status	Description
Normal working status	Blue	breathing light	Energy Mode
	Green	breathing light	Relax Mode
	Cyan	breathing light	Sleep Mode
	Cyan	on light	Music Mode

Music indicator light			
	Colour	Status	Description
Normal working status	Cyan	breathing light	Bluetooth connected
Search for new Bluetooth devices	Cyan	on light	Bluetooth not connected
Search for new Bluetooth devices	Blue	flash light	Search for new Bluetooth devices

Intermediate indicator light			
	Colour	Status	Description
Normal working status	Orange	on light	Heating on
	Blue	on light	Heating off
Charging status	Orange	breathing light	Charging
	Blue	on light	Fully charged

3

**Contents**

Important	01
Important Safety Precautions	01
Product Diagram	02
Usage Method	03
Before use	03
Indicator light description	03
At use	04
Maintenance	05
How to clean	05
Storage	05
Error Diagnosis	06
Product Specifications	07
Warranty & Customer Support	07

**Maintenance**

**How to clean**

- Surface soiling can be removed with a clean rag;
- For deep dirt, wet the cloth with neutral detergent and gently wipe it off.
- Do not pour water to the product
- Leave it to dry naturally.

**Storage**

Please place the product inside of the storage bag with its original package and store it in a safe place.

Do not put heavy weights on top of the product.

Please take moisture, rodent and pollution prevention measures before storing the product.

For product in more than 3 months of storage, please recharge before use.

Do not store the product in places with direct sunlight, or corrosive gas.

5

**Error Diagnosis**

Error	Reason	Solution
Can not turn on.	Insufficient battery.	Please recharge.
Air-pressure strength is weak.	The strap is too loose.	Adjust the length of the strap, make the massager comfortable to wear
Stopped working during operation.	Massage timer is off.	Press the Power button to reactivate the product.
	Battery is ran out.	Recharge the battery.
Cannot connect Bluetooth	Mistakenly touched the music button. Bluetooth is turned off	Long press the power button for 2 seconds to turn off, and the Bluetooth function will be restored after restarting.
Bluetooth playback music sound is low	Mobile phone playback volume is low	Adjusting the phone volume on your phone

6

**Product Specifications**

Product name	Eye Massager
Product model	MGE-2304
Product power	4W
charging voltage	5V
battery capacity	1200mAh
Product size	310×90×82mm
Net weight	242g

**Warranty & Customer Support**

If you need help regarding product issues, after-sales service, or other relevant problems, please contact us via the email on the cover.

\*Note: Products purchased directly from NAIPO benefit from our after-sales service. If the purchases are made from local retail stores, we recommend you contact them for warranty issues.

7

**At use**

Buttons	
Power Button	1. long press the power button 2 seconds, ON/OFF
	2. Default power on into the "Energy Mode", short press the power button to switch the massage mode, in order of "Energy Mode", "Relax Mode", "Sleep Mode", "Music Mode", "Music Mode" can only turn on heating, no massage function.
	3. "Energy Mode" Default low heating, automatically play built-in music, double-click for high heating, double-click again to turn off heating.
	4. "Relax Mode" Default high heating, automatic playback of built-in music, double-click to turn off heating, Double click again to turn on low heating.
	5. "Sleep Mode" Default low heating, automatically play the built-in music, double-click for high heating, double-click again to turn off heating.
	6. "Music Mode" Default low heating, automatically play built-in music, double-click for high heating, double-click again to turn off heating.
Music button	1. The first time to connect Bluetooth, long press the music button 2 seconds, search for new Bluetooth devices, the Bluetooth name for this product is "NAIPO". If need to disconnect current Bluetooth device, Please long press the music button for 2 seconds.
	2. Double-click the music button (press twice within 1 seconds) to turn on/off the music.
	3. Short press the music button to switch next music.

4